



NB-IFU-00 Issued 15/12/2013



NBlade

Disposable Neonatal Laryngoscope Blade

Instruction for use

Warnings and Cautions

- The blade is packed ready for use.
- Don't use if package is opened or damaged
- The blade supposed to be used for single patient
- The blade is non sterile but it could be sterilize as packed by EtO if local regulations required it.
- It is the user's responsibility to read and to understand these instructions prior to use of
- Peak Medical laryngoscope blades and handles.
- Tracheal intubation requires specialized training. Only personnel who have received such training should be authorized to use laryngoscopes.
- Do not attempt to use any laryngoscope unless pre-use checks as described below have been carried out successfully.
- Store in dry clean condition away from heat and light.

Pre-use checks prior to commencing an intubation procedure:

- Check handle for sharp edges or flaws that may have resulted from mechanical damage.
- Ensure blade is securely and correctly clicked into place on the handle.

Operating Instruction

- Attach laryngoscope blade to the handle.
- Ensure blade is fitted firmly on the handle and clicked into the place.
- Don't apply excessive force during intubation
- Discard after use. Don't sterilize.

Cleaning, disinfection and sterilization

- Don't sterilize after use.
- Discard after use.
- Dispose of NBlade according to standard hospital procedure



Peak Medical Ltd. 10 Giborey Israel 10 Netanya 4250410 Israel



Obelis a.a Bd General Wahis 53 ,1030 Brussels. Tel: +3227325954, Fax: 3227326003, e-mail: mail@obelis.net

