

VNBlade

Disposable Neonatal Video Laryngoscope Blade

Instruction for use

Warnings and Cautions

- The blade is packed ready for use.
- Don't use if package is opened or damaged
- The blade supposed to be used for single patient
- The blade is non sterile but it could be sterilize as packed by EtO if local regulations required it.
- It is the user's responsibility to read and to understand these instructions prior to use of Peak Medical laryngoscope blades and handles.
- Tracheal intubation requires specialized training .Only personnel who have received such training should be authorized to use video laryngoscopes.
- Do not attempt to use any laryngoscope unless pre-use checks as described below have been carried out successfully.
- Store in dry clean condition away from heat and light.

Pre-use checks prior to commencing an intubation procedure:

- Choose handle and blade size in accordance with weight of newborn (make sure that size similar for both).
- Check handle for sharp edges or flaws that may have resulted from mechanical damage.
- Ensure blade is securely and correctly clicked into place on the handle before application.

Operating Instruction

- Attach laryngoscope blade to the handle.
- Ensure blade is fitted firmly on the handle and clicked into the place.
- Check quality of image on monitor, if it isn't adequate exchange the blade and check again.
- Don't intubate till the image will acceptable.
- Discard after use. Don't sterilize.
- Dispose of blade according to standard hospital procedure.



Manufactured by: Peak Medical Ltd. 14 Givaty Str.P.O.Box.4044 Netanya 4214001 Israel, 2016

						
Manufacturer name	Product non	Special operation	Date by which device	Device is for	Batch number	EC representative
and address	sterile	instructions	should be used in safety	single use		name and address