

NOX-S starter guide

Important!: surfactant administration procedure and type of catheter should conform manufacturer recommendations , local practice and regulations.


Video Laryngoscope Peak VN with attachment NOX-S is intended to facilitate intubation only!

Preparations before application

Choose VNScope handle according to the weight of the baby,


Insert VNBlade of a suited size and check if it is fixed properly.

Prepare the required catheter (maximal outer diameter of the catheter is Fr5 or 1.67mm) and NOX-S attachment.

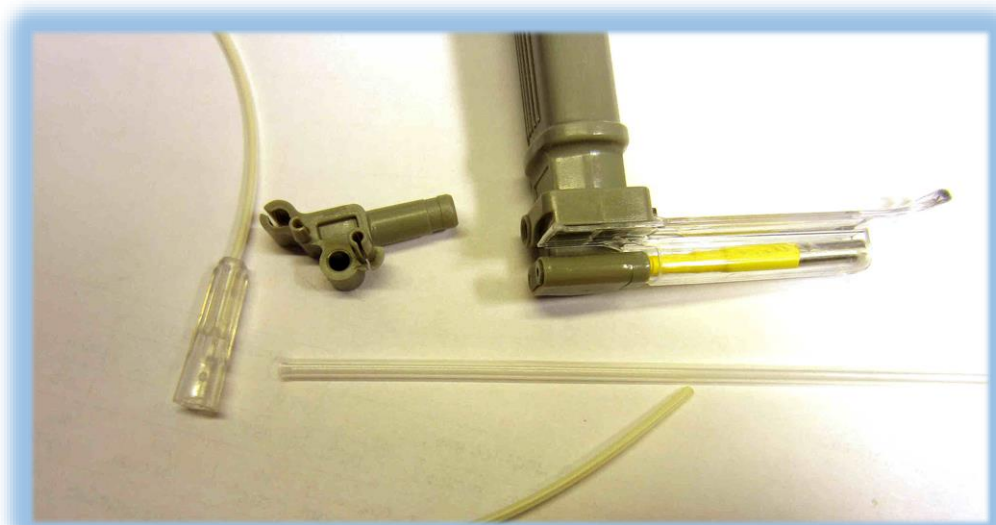


NOX-S starter guide

Attach NOX-S to VNScope handle properly and check that gas outlet nozzle is directed along the VNBlade axis.

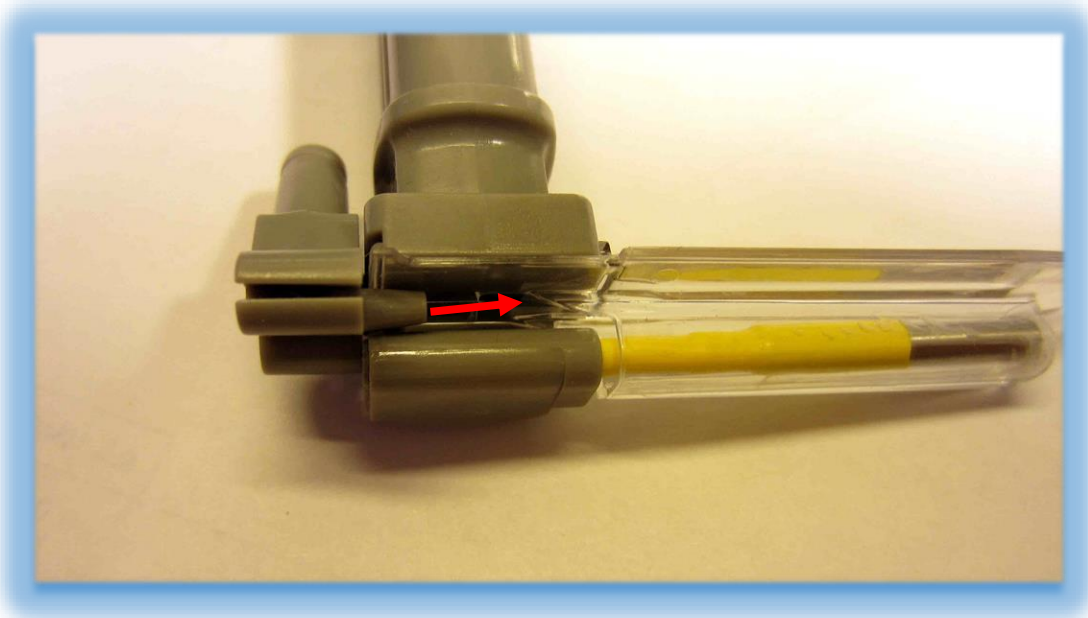


Insert guiding tube in such a way that its rounded tip protrudes from tip of the camera by 3-5 mm but does **not protrude from the tip of the blade**

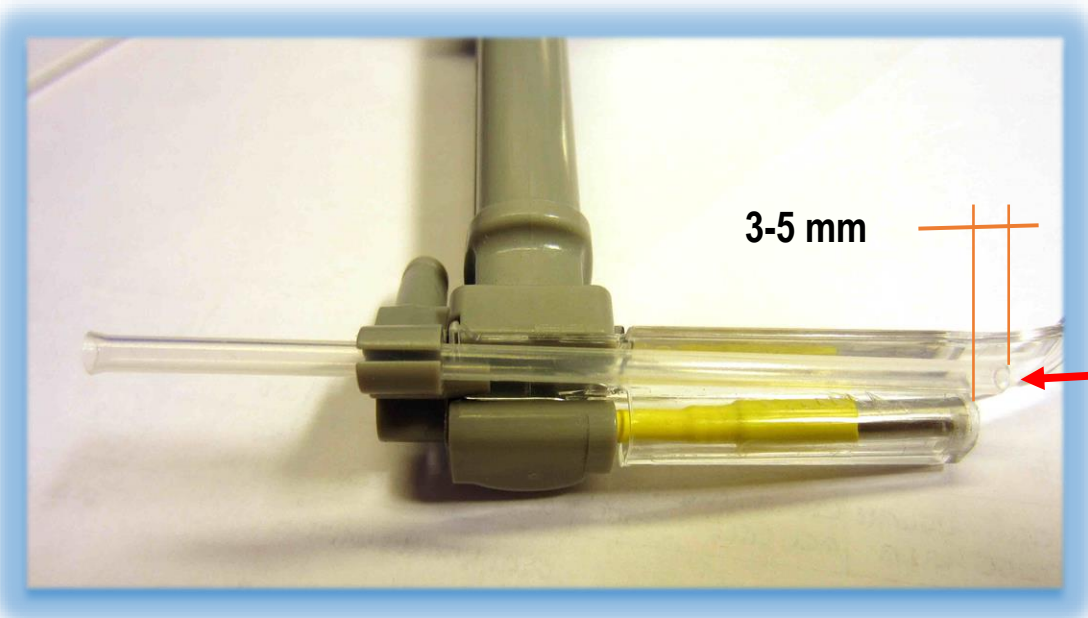

NOX-S starter guide

Insert the catheter into the guiding tube

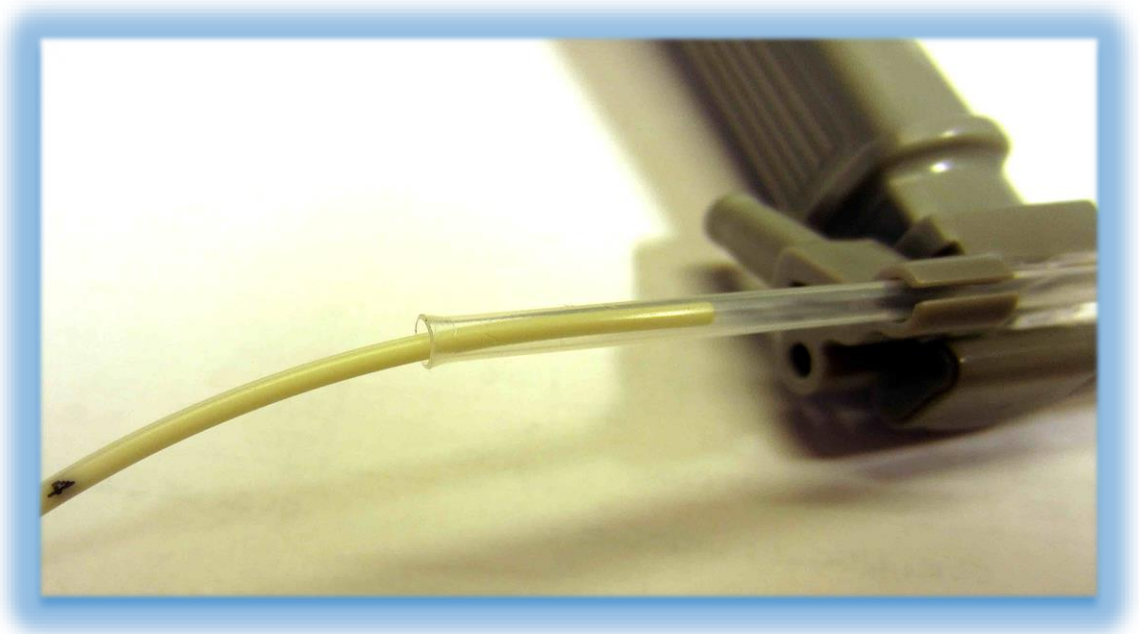

Provide that the tip of the catheter slightly protrudes from the guiding tube



NOX-S starter guide

Check on monitor if the tip of the catheter is visible



Proceed further according to established procedure for surfactant administration.

Remark: NOX-S could be connected to a gas supply line to provide continuous insufflation during intubation.