



Compact



Disposable Laryngoscope Blade with Handle

Instruction for use

Warnings and Cautions

- The Laryngoscope is packed ready for use.
- Don't use if package is opened or damaged.
- The Laryngoscope supposed to be used for single patient
- It is the user's responsibility to read and to understand these instructions prior to use of Peak Medical laryngoscope blades and handles.
- Tracheal intubation requires specialized training .Only personnel who have received such training should be authorized to use laryngoscopes.
- Do not attempt to use any laryngoscope unless pre-use checks as described below have been carried out successfully.
- Store in dry clean condition away from heat and light.

Pre-use checks prior to commencing an intubation procedure:

- Check Laryngoscope for sharp edges or flaws that may have resulted from mechanical damage.
- Ensure blade is securely and correctly clicked into place on the handle.
- Switch On LED (see below) to check light intensity. No light indicates Laryngoscope damage.

Operating Instruction

- Open laryngoscope blade by rotating it.
- Ensure blade is fitted firmly on the handle and clicked into the place.
- Switch On LED
- Discard after use. Don't sterilize.
- Dispose of blade according to standard hospital procedure.

Cleaning, disinfection and sterilization

- Don't sterilize after use.
- Discard after use.
Dispose of Compact Laryngoscope according to standard hospital procedure



Manufactured by: Peak Medical Ltd. 10 Giborey Israel 10 Netanya 4250410 Israel,



Obelis a. Bd General Wahis 53 ,1030 Brussels. Tel: +3227325954, Fax: 3227326003, e-mail: mail@obelis.net

