



DNS-IFU-00 Issued 20/1/2016



DNScope

Disposable Neonatal Laryngoscope Handle

Instruction for use

Warnings and Cautions

- The handle is packed ready for use.
- Don't use if package is opened or damaged.
- The handle supposed to be used for single patient
- It is the user's responsibility to read and to understand these instructions prior to use of Peak Medical laryngoscope blades and handles.
- Tracheal intubation requires specialized training. Only personnel who have received such training should be authorized to use laryngoscopes.
- Do not attempt to use any laryngoscope unless pre-use checks as described below have been carried out successfully.
- Store in dry clean condition away from heat and light.

Pre-use checks prior to commencing an intubation procedure:

- Check handle for sharp edges or flaws that may have resulted from mechanical damage.
- Ensure blade is securely and correctly clicked into place on the handle.
- Switch On LED by rotating a knob of the handle clockwise to check light intensity. Don't apply excessive force.
- No light indicates handle damage.
- Switch Off the LED by rotating the knob counterclockwise.
- Check magnifying glass for cleanness
- Check magnifying glass for proper positioning without free play.
- If it is required, attach NOX or NOX-S in accordance with corresponding IFU

Operating Instruction

- Attach laryngoscope blade to the handle.
- Ensure blade is fitted firmly on the handle and clicked into the place.
- If, NOX or NOX-S is attached apply it in accordance with corresponding IFU
- Switch On LED by rotating the knob of the handle clockwise approximately 180Deg. **Don't apply excessive force**
- Apply magnifying glass if needed by rotating it to a proper position.
- After intubation switch Off the LED by rotating it counterclockwise.
- Discard after use. Don't sterilize.
- Dispose of blade according to standard hospital procedure.

Cleaning, disinfection and sterilization

- Don't sterilize after use.
- Discard after use.
- Dispose of DNScope and blade and NOX according to standard hospital procedure



Manufactured by: Peak Medical Ltd, 10 Giborey Israel 10 Netanya 4250410 Israel,



Obelis a. Bd General Wahis 53 ,1030 Brussels. Tel: +3227325954, Fax: 3227326003, e-mail: mail@obelis.net

